

EndoWrist™ Stapler

For the da Vinci® Xi & X Surgical System

The EndoWrist Stapler offers a surgeon-controlled, fully wristed stapling instrument and SmartClamp™ Technology.

Instrument Information



The EndoWrist Stapler for the da Vinci Xi System is an endoscopic linear stapler. The EndoWrist Stapler provides fully wristed articulation in both horizontal and vertical directions and is controlled by the surgeon through the da Vinci Xi Surgeon Console.

SmartClamp technology provides intraoperative feedback to the surgeon and reduces the guesswork

normally associated with stapling. For each reload color, SmartClamp technology detects whether the jaws can adequately close on the target tissue for the given staple height. If not, the EndoWrist Stapler will notify the user.

The EndoWrist Stapler System also prevents firing when it detects that no reload is installed, or that a spent reload is installed, and notifies the surgeon and OR staff.

EndoWrist Stapler for da Vinci Xi System Start Kit (PN 381251)

The Starter Kit provides the necessary components, except for the reloads, for the first 10 cases. All reloads must be ordered separately. The Starter Kit provides 100 firings with 2 EndoWrist Stapler instruments. It also includes the necessary training products to begin using the Stapler.

Quantity	EndoWrist Stapler System Starter Kit Contents	
2	EndoWrist Stapler 45 Instrument	Reusable
1 Box	EndoWrist Stapler Sheath (10 per box)	Disposable procedural item
1 Box	12 mm & Stapler Cannula Seal (10 per box)	Disposable procedural item
2 Boxes	EndoWrist 12 - 8 mm Cannula Reducer (6 per box)	Disposable procedural item
2	EndoWrist Stapler Cannula Kit	Accessory

EndoWrist Stapler for da Vinci System Reloads

Reloads are ordered separately from the Starter Kit. The hospital determines the appropriate reload color and orders the amount based on procedural use and surgical volume.

Product Code	Description	Uses	Units Per Box
48645W	EndoWrist Stapler 45 White Reload, 2.5 mm	1 firing	12
48645B	EndoWrist Stapler 45 Blue Reload, 3.5 mm	1 firing	12
48445G	EndoWrist Stapler 45 Green Reload, 4.3 mm	1 firing	12
48630M	EndoWrist Stapler 30 Gray Reload, 2.0 mm	1 firing	12
48630W	EndoWrist Stapler 30 White Reload, 2.5 mm	1 firing	12
48630B	EndoWrist Stapler 30 Blue Reload, 3.5 mm	1 firing	12
48630G	EndoWrist Stapler 30 Green Reload, 4.3 mm	1 firing	12

EndoWrist Stapler for da Vinci System Reposables

Product Code	Description	Uses	Units Per Box
470298	EndoWrist Stapler 45 Instrument	50 firing	1
470545	EndoWrist Stapler 45 Curved-Tip Instrument	50 firing	1
470430	EndoWrist Stapler 30 Instrument	50 firing	1
470530	EndoWrist Stapler 30 Curved-Tip Instrument	50 firing	1

EndoWrist Stapler for da Vinci System Disposables

Product Code	Description	Uses	Units Per Box
410370	EndoWrist Stapler Sheath	One procedure	10
470380	12 mm & Stapler Cannula Seal	One procedure	10
470381	EndoWrist 12 - 8 mm Cannula Reducer	One procedure	6

EndoWrist Stapler for da Vinci System Accessories

Product Code	Description	Uses	Units Per Box
470375	EndoWrist 12 mm & Stapler Cannula	NA	1
470376	EndoWrist 12 mm & Stapler Blunt Obturator	NA	1
470395	EndoWrist 12 mm & Stapler Bladeless Obturator	NA	1
470389	EndoWrist 12 mm & Stapler Cannula, Long	NA	1
470390	EndoWrist 12 mm & Stapler Blunt Obturator, Long	NA	1
470396	EndoWrist 12 mm & Stapler Bladeless Obturator, Long	NA	1
381215	EndoWrist Stapler Release Kit	NA	1

Warranty Information

The EndoWrist Stapler System is sold, installed, and serviced exclusively by Intuitive Surgical.

www.intuitivesurgical.com • www.davincisurgery.com

To order, please call Intuitive Surgical Customer Service at 1-877-408-3872 or 1-800-876-1310 (US).

IMPORTANT SAFETY INFORMATION

The EndoWrist Stapler 45 and Stapler 30 Instruments, Stapler 45 and Stapler 30 Reloads and other Stapler Accessories are intended to be used with the da Vinci Xi Surgical System for resection, transection and/or creation of anastomoses in General, Thoracic, Gynecologic and Urologic surgery. The devices can be used with staple line or tissue buttressing material (natural or synthetic).

The EndoWrist Stapler 45 and Stapler 30 Instruments, Stapler 45 and Stapler 30 Reloads and other Stapler Accessories are class 1s, 2a and 2b medical devices CE marked (CE0543) under the European Medical Devices Directive (93/42/EEC), manufactured by Intuitive Surgical, Inc. Refer to Instructions For Use before use.

EUROPE	DEUTSCHLAND	FRANCE	UNITED KINGDOM
Intuitive Surgical Sàrl	Intuitive Surgical GmbH	Intuitive Surgical SAS	Intuitive Surgical Ltd
1 Chemin des Mûriers	Pelmanstrasse 46	41-43 Rue Perier	16-18 Station Road, Chapeltown
1170 Aubonne, CH	45131 Essen, DE	92120 Montrouge, FR	S35 2XH Sheffield, UK
VAT #: CHE-113.407.162 VAT	MWST #: DE200038727	TVA #: FR 64 417 889 177	VAT #: GB943 2213 49
	Amtsgericht Essen, HRB 13476	SIRET 417 889 177	Company registration No 6727762
		Capital Social 37000€	
		RCS Nanterre Code APE 4646Z	