

# SureForm™ from da Vinci® Stapling

## Achieve more with your stapler.

SureForm™ is a new breed of stapler. At its core is SmartFire technology, which monitors tissue compression before and during firing. With more than 1,000 measurements per second,<sup>1</sup> SmartFire makes automatic adjustments to optimize the staple line and prevent tissue damage across a range of tissue thicknesses.<sup>1</sup> A 120° cone of articulation gives you the freedom to approach tissue where it naturally lies, potentially reducing the need to pull it towards the stapler.<sup>1</sup> When you choose SureForm staplers, you can reduce uncertainty and increase stapling performance like never before.<sup>1</sup>


SureForm staplers are available in **45 mm** and **60 mm** staple line lengths. SureForm™ 45 mm and 60 mm reloads include six rows of staples, and introduce **black reloads** to the da Vinci Stapling portfolio for the first time.

### SureForm 60



60 mm staple line / 55 mm cut line  
An anvil centric design  
Straight-tip anvil  
Operates through a 12 mm cannula  
12 fires per device  
Single patient use

### SureForm 45 & SureForm 45 Curved-Tip

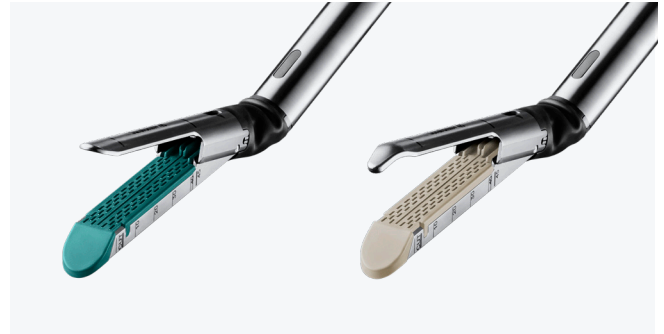

45 mm staple line / 40 mm cut line  
An anvil centric design  
Straight-tip or curved-tip anvil  
Operates through a 12 mm cannula  
12 fires per device  
Single patient use

### SureForm 60 reloads



White, Blue, Green, and Black reloads  
6 rows of staples  
New knife with every reload

### SureForm 45 reloads



Gray, White, Blue, Green, and Black reloads  
6 rows of staples  
New knife with every reload

1. Results from internal testing on file.

# Purchasing information

## SureForm™ staplers

ITEM CODE	PRODUCT DESCRIPTION	USES	UNIT/BOX
480460	Stapler, SureForm 60 Instrument	Single use	6 devices
480445	Stapler, SureForm 45 Instrument	Single use	6 devices
480545	Stapler; SureForm 45 Curved-Tip Instrument	Single use	6 devices

## SureForm™ reloads

ITEM CODE	PRODUCT DESCRIPTION	USES	UNIT/BOX
48360W	Reload, SureForm 60, 2.5 White, 6-Row	One firing	12 units
48360B	Reload, SureForm 60, 3.5 Blue, 6-Row	One firing	12 units
48360G	Reload, SureForm 60, 4.3 Green, 6-Row	One firing	12 units
48360T	Reload, SureForm 60, 4.6 Black, 6-Row	One firing	12 units
48345M	Reload, SureForm 45, 2.0 Gray, 6-Row	One firing	12 units
48345W	Reload, SureForm 45, 2.5 White, 6-Row	One firing	12 units
48345B	Reload, SureForm 45, 3.5 Blue, 6-Row	One firing	12 units
48345G	Reload, SureForm 45, 4.3 Green, 6-Row	One firing	12 units
48345T	Reload, SureForm 45, 4.6 Black, 6-Row	One firing	12 units

## 12 mm & stapler disposables

ITEM CODE	PRODUCT DESCRIPTION	USES	UNIT/BOX
470380	12 mm & Stapler Cannula Seal	1 procedure	Box of 10
470381	12-8 mm Reducer	1 procedure	Box of 6

## 12 mm & stapler access system

ITEM CODE	PRODUCT DESCRIPTION	USES	UNIT/BOX
470375	12 mm & Stapler Cannula (100 mm)	N/A	1 unit
470376	12 mm & Stapler Blunt Obturator	N/A	1 unit
470389	12 mm & Stapler Cannula, Long (150 mm)	N/A	1 unit
470390	12 mm & Stapler Blunt Obturator, Long	N/A	1 unit
470395	12 mm & Stapler Bladeless Obturator	N/A	1 unit
470396	12 mm & Stapler Bladeless Obturator, Long	N/A	1 unit

### To contact a representative or receive additional information:

Visit [www.intuitive.com](http://www.intuitive.com) or call Intuitive Surgical Customer Service in the U.S. at 1.877.408.3872, in Europe at +41.21.821.2020 or +800.0821.2020 or in the rest of the world, 1.408.523.2100

---

### **Important safety information**

Serious complications may occur in any surgery, including da Vinci® surgery, up to and including death. Examples of serious or life-threatening complications, which may require prolonged and/or unexpected hospitalization and/or reoperation, include but are not limited to, one or more of the following: injury to tissues/organs, bleeding, infection and internal scarring that can cause long-lasting dysfunction/pain.

Risks specific to minimally invasive surgery, including da Vinci® surgery, include but are not limited to, one or more of the following: temporary pain/nerve injury associated with positioning; a longer operative time, the need to convert to an open approach, or the need for additional or larger incision sites. Converting the procedure could result in a longer operative time, a longer time under anesthesia, and could lead to increased complications. Contraindications applicable to the use of conventional endoscopic instruments also apply to the use of all da Vinci instruments.

For Important Safety Information, indications for use, risks, full cautions and warnings, please also refer to [www.intuitive.com/safety](http://www.intuitive.com/safety).

Individuals' outcomes may depend on a number of factors, including but not limited to patient characteristics, disease characteristics and/or surgeon experience.

The Intuitive Surgical Stapler SureForm 60, Stapler SureForm 60 reloads, and other stapler accessories are intended to be used with a compatible da Vinci Surgical Systems for resection, transection, and, or creation of anastomoses in General, Thoracic, Gynecologic, Urologic, and Pediatricsurgery. The device can be used with staple line or tissue buttressing material (natural or synthetic).

The Intuitive Surgical SureForm 45 Stapler, SureForm 45 Reloads and other stapler accessories are intended to be used with a compatible da Vinci Surgical System for resection, transection, and, or creation of anastomoses in General, Thoracic, Gynecologic, Urologic, and Pediatric surgery. The device can be used with staple line or tissue buttressing material (natural or synthetic).

© 2019 Intuitive Surgical, Inc. All rights reserved. Product and brand names/logos are trademarks or registered trademarks of Intuitive Surgical or their respective owner. See [www.intuitive.com/trademarks](http://www.intuitive.com/trademarks).

PN 1057632-US Rev B 07/2019