

PHYSICIAN	DEPARTMENT	HOSPITAL	TASK	MONOPOLAR BIPOLAR	MONOPOLAR SETTINGS		BIPOLAR SETTINGS
				Instrument	Cut	Coag	
Dr. Clayton Lau	Urology	City of Hope Medical Center	Dissection	MCS MBF or FBF	Auto Effect 4 Power Limit 80W	Swift Effect 4 Power Limit 80W	Autostop Off Effect 4 Power Limit 80W
			Tumor Resection		Auto Effect 5 Power Limit 80W	Swift Effect 5 Power Limit 80W	Autostop Off Effect 4 Power Limit 80W
Dr. Warren Oberle	Urology	Wellstar Cobb Hospital	Skin Incision	MCS MBF or FBF	Auto Effect 4 Power Limit 80W	N/A	N/A
			Dissection		N/A	Classic Effect 1 Power Limit 60W	Autostop Off Effect 4 Power Limit 80W
Dr. Kathy Huang	Gynecology	NYU Langone Medical Center	Dissection	MCS LBG	N/A	Swift Effect 3 Power Limit 25W	Autostop Off Effect 4 Power Limit 35W
			Thick tissue dissection (EX:Uterine & Fibroid tissue)		N/A	Swift Effect 3 Power Limit 35W	Autostop Off Effect 4 Power Limit 35W

Important Safety Information

For Important Safety Information, indications for use, risks, full cautions and warnings, please refer to www.davincisurgery.com/safety and www.intuitivesurgical.com/safety.